



Gettig™ Guard Safety Needle

Instructions for Use

- Remove the Gettig™ Guard Safety Needle from the package and attach to any standard syringe.

- Press **release tab “a”** and pull the guard down firmly over the syringe to expose the needle.


- Draw the medication, then push the guard up to cover the needle. An audible click will confirm the needle is safe for transport to the patient site.

- To give injection, press **release tab “a”** and pull the guard down firmly over the syringe to expose the needle.

- Administer the injection.

- Immediately push the guard up to cover the needle. Press **lock tab “b”** to permanently lock the guard in place.

- Discard the needle and syringe in an approved sharps container.

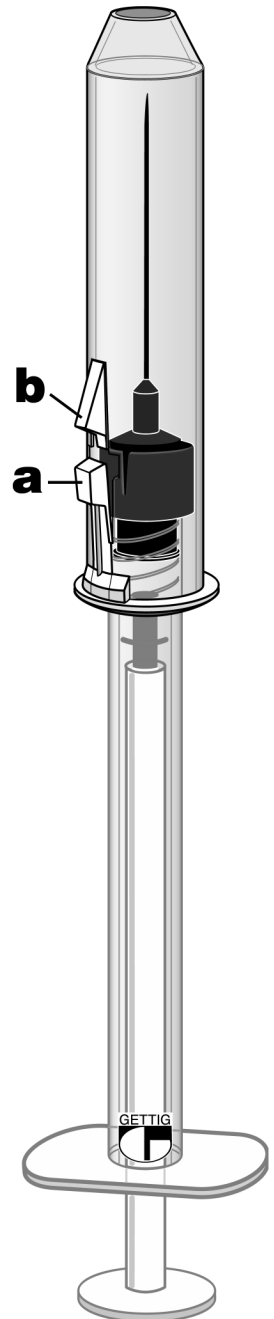

Gettig Pharmaceutical Instrument Company
A Division of Gettig Technologies Inc.
(814) 422-8892
www.gettig.com